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Magnet field therapy-System Magneto Fit MF 300 und MF 400

Table of Contents-

Summary.....	7
Before operation.....	7
Instructions for using.....	7
Contraindications	8
Operation of Magneto Fit MF 300 and MF 400	8
Operation of Magneto Fit MF 300	8
Operation of Magneto Fit MF 400	9
Additional functions of the Magneto Fit MF 400 in comparison to MF 300.....	10
Cleaning, maintenance and disposal	10
Guarantee.....	10
Technical specifications, scope of delivery and accessory.....	10

Before operation

Dear customer,

We enjoy that you bought a product from DAVITA® and congratulate you to your choice. In this user manual particularly the functions of the Magnetic field system MF 300 and MF 400 are described in detail.

Safety instructions

- ✚ Place the therapy unit on a compact and plane surface.
- ✚ Check by mains operation if the power supply has the necessary 110 – 240 V by 50 – 60 Hz.
- ✚ The control unit should only be used with the wall power supply which belongs to it. The operation unit should only be connected to the appropriate treatment mat.
- ✚ This therapy unit should only be used inside. Avoid rain or much humidity.
- ✚ Do not place a case with liquid on or beside the unit.
- ✚ The magnetic field therapy has not the same effect for all people. Therefore every user should determinate the appropriate duration of the application by themself. Many people use it approx. 30 minutes. The same is valid for the frequency. Start with a lower frequency for example 5 Hz. Increase the frequency slowly in accordance to your feeling. With the so determinated values further applications can be done.
- ✚ Don't buckle the cables too much and do not expose the cables to heat or chemicals. If a cable is defect, pull it out of the unit and let it check by authorized qualified personnel.
- ✚ Do not open the unit. All service has to be done by authorized qualified personnel.
- ✚ Take off all metallic objects (jewels, watches etc.) before doing the treatment.

Before using the instrument for the first time the safety instructions have to be read. To reduce the risk of electric shocks, burns, fire, or other injuries, the following safety instructions have to be observed:

- ✚ The light alarm clock LiteClock is not allowed to be placed in water or near heated surfaces.
- ✚ To prevent inadvertent contact with water, do not use the LiteClock while bathing and do not pick up the adapter or cord if the hands are wet.

Contraindications

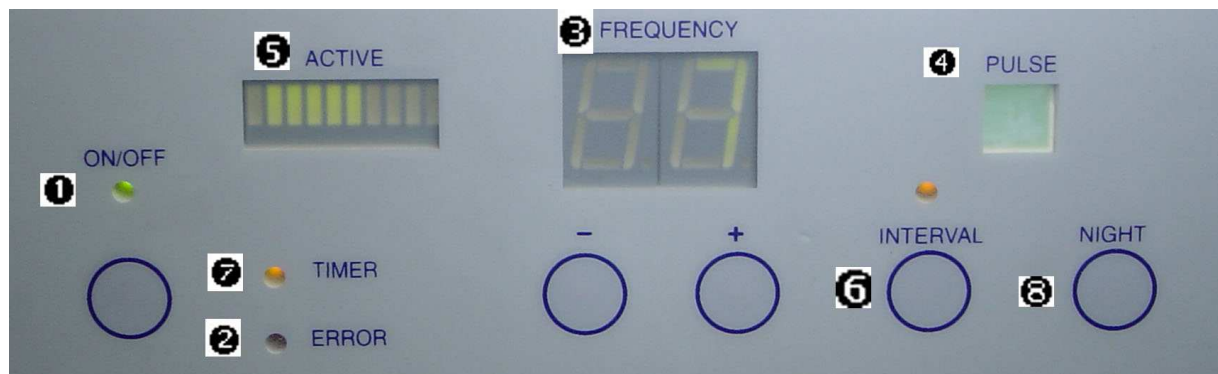
Warning: The magnetic field therapy is not allowed to use by patients:

- ✚ with a pacemaker (ad hoc decision) • with an ICD (implantated cardiofibrillator)
- ✚ with cancer (for cases in which the illness can be dated back for less than 5 years) • in the childbearing period
- ✚ with bacterial, inflammable athropathy•
- ✚ with massive, feverish infectious disease

In these cases please contact your physician .

Operation of Magneto Fit MF 300 und MF 400

Operation of Magneto Fit MF 300

Circuit points: ① Wall power supply, ② Magnetic field therapy mat

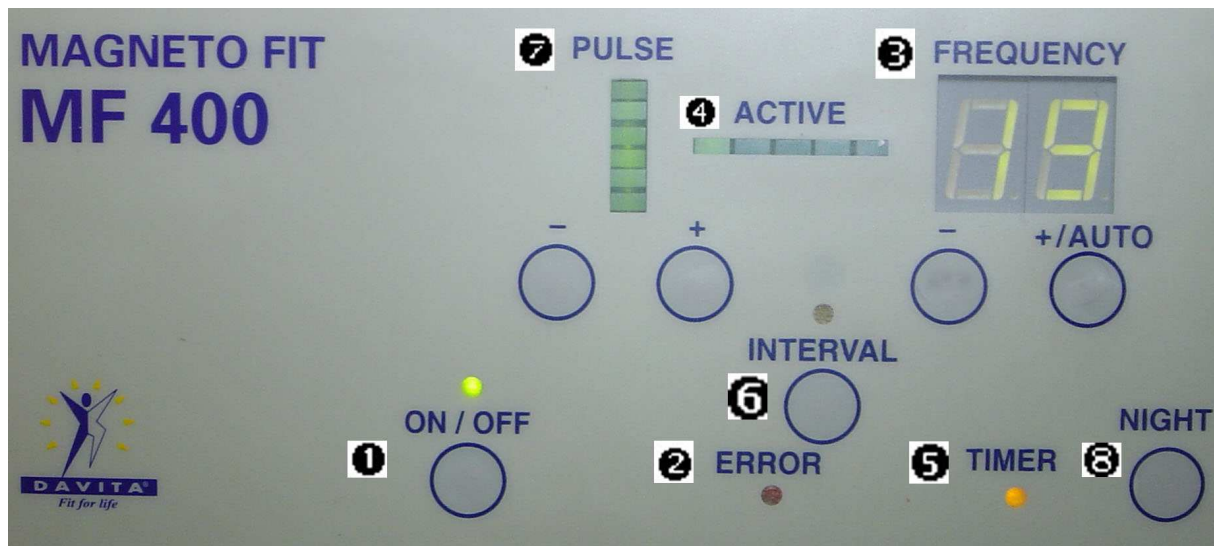
- ① **ON/OFF:** By pushing the button the unit can be switched on or off. After switching on the unit the ON/OFF-LED begins to shine.
- ② **ERROR:** This LED shines, if the mat is not or not correct connected with the control unit. The unit can not be used.
- ③ **FREQUENCY:** By pushing the „+“ button the frequency can be increased up to 19 Hertz. By pushing the „-“ button the frequency can be decreased until 0 Hertz. If 0 Hertz is reached the control unit is deactivated.
- ④ **PULSE:** Flash in the rhythm of the adjusted frequency.
- ⑤ **ACTIVE:** If the control unit is operating, the LEDs of the display will be powered one after another (Running light). The speed of the running light is independent from the adjustment of the frequency.
- ⑥ **INTERVAL:** By pushing the button the LED will be activated. There is a change in the function of the unit. The control unit will be switched on for 6 seconds and after that switched off for 3 seconds (Rest) After that the process starts again. By pushing the button again the function can be deleted. Some scientists have the opinion, that the body can assimilate the energy of the magnetic fields so optimal.
- ⑦ **TIMER:** If the frequency is adjusted higher than 4 hertz the timer switches on and the LED begins to shine. The Timer will be on for 30 minutes. After this time the units switch off. The frequency will be adjusted to 0 Hertz.

- 8 NIGHT:** By pushing the button all displays will be cleared. Only the small LED in the frequency display is shining. The function of the unit will be unaffected. By pressing any button the displays will be switched on again.

Storage of the operation parameters:

By pushing the ON/OFF-button the frequency will be stored. If the unit will be switched on again these parameters will be imported and the unit works with these parameters. So it is possible to switch on the unit with an usual clock timer and the unit starts with the stored parameter.

Operation of Magneto Fit MF 400

Circuit points: 1 Wall power supply, 2 Magnetic field therapy mat

- 1 ON/OFF:** By pushing the button the unit can be switched on or off. After switching on the unit the ON/OFF-LED begins to shine.
- 2 ERROR:** This LED shines, if the mat is not or not correct connected with the control unit. The unit can not be used.
- 3 FREQUENCY:** By pushing the „+“ button the frequency can be increased up to 19 Hertz. By pressing the „-“ button the frequency can be decreased until 0 Hertz. If 0 Hertz is reached the control unit is deactivated.
FREQUENZROTATION: If the „+“ button will be pushed a little longer at 19 hertz the frequency rotation function will be activated. Here the single frequencies are rotating active for 5 seconds. Many users like this function very much. By pushing the „-“ button this modus ca be left.
- 4 ACTIVE:** If the control unit is operating, the LEDs of the display will be powered one after another (Running light). The speed of the running light is independent from the adjustment of the frequency.
- 5 TIMER:** If the frequency is adjusted higher than 4 hertz the timer switches on and the LED begins to shine. The Timer will be on for 30 minutes. After this time the units switch off. The frequency will be adjusted to 0 Hertz.
- 6 INTERVAL:** By pushing the button the LED will be activated. There is a change in the function of the unit. The control unit will be switched on for 6 seconds and after that switched off for 3 seconds (Rest) After that the process starts again. By pressing the button again the function can be deleted. Some scientists have the opinion, that the body can assimilate the energy of the magnetic fields so optimal.
- 7 PULSE:** By pushing the „+“ or „-“ button the intensity will be increased or decreased. Here exists the possibility to variate the magnetic active time between 15% and 90%. The application time can be considerably shortened. Every illuminated line in the LED means an increase of 15%.
- 8 NIGHT:** By pushing the button all displays will be cleared. Only the small LED in the frequency display is shining.

The function of the unit will be unaffected. By pressing any button the displays will be switched on again.

Storage of the operation parameters:

By pushing the ON/OFF-button the intensity, frequency and the frequency rotation modus will be stored. If the unit will be switched on again these parameters will be imported and the unit works with these parameters. So it is possible to switch on the unit with an usual clock timer and the unit starts with the stored parameter.

Additional functions of Magneto Fit MF 400 in comparison to MF 3

-  The intensity can be adapted individually.
-  Frequency rotation modus: The individual frequencies are rotation, each is active for 5 seconds.
-  Expanded storage function: The intensity, the frequency and the frequency rotation modus are stored.

Cleaning, maintenance and disposal

A special maintenance is not necessary for this device. Before cleaning the unit has to be switched off and all plugs have to be pulled out. For cleaning the unit a soft, wet rag and a mild household cleaning agent can be used. The cleaning agent should not be biting or rubbing. The magnetic therapy mat is carefree.

Packaging: DAVITA® joint to the system „Grüner Punkt“. All parts of the packaging are recycable materials and environmentally acceptable to dispose. Batteries and electronic units which are signed with the crossed wheelie bin symbol are not allowed to throw into the domestic waste. There is the commitment to dispose such electro- and electronic „Altgeräte“ separately. The communal public authority which is responsible for the disposal gives information regarding the possibilities of a regulated disposal.

Note STK: In accordance to the “Medizinprodukte-Betreiberverordnung (MP BetreibV) § 6 (Sicherheitstechnische Kontrollen)“ the operator is obliged to let carry out a safety technical control every 2 years. We advise to carry out the controls by DAVITA® or from a special authorized qualified personnel. The unit which should be maintained can be dispatched directly to the producer or to the distributor, where it has been sold.

Guarantee

DAVITA® guarantees for this instrument for a period of 24 months after date of sale. The guarantee is valid for material defects and manufacturing faults and includes the provision of spare parts as well as labour. Guarantee expires if the instrument has been damaged, or used improperly or unauthorized intervention was carried out. Keep the purchase receipt as proof and submit it together with your claim for guarantee. DAVITA® will only fulfil the guarantee upon receipt of your purchase proof.

Technical specifications, scope of delivery and accessory

Wall power supply

Model: MKD 48 - 1201000GS
Input: 230 V AC 50 Hz
Output: 12 V DC 1000 mA

Control unit

Type: MF 300 and MF 400, CE 1275
Power consumption: 12 V DC / 8,5 VA
Output: 12 V / 6,5 W
Frequency: 1 – 19 Hz
Magnetic induction: 0,56 mT max.
Automatic switch off:
From 1 – 4 Hz no switch off
From 5 – 19 Hz switch off after approx. 30 minutes

Applicator

Mat: 190 x 85 cm
Power consumption:
12 V pulsed 0,7 A
Field intensity: 0,56 mT max.

Weight of the complete system: approx. 4,2 kg

Scope of delivery: Control unit, mat, wall power supply and detail user manual.

Magneto Fit MF 300: Art. No.: 60009

Magneto Fit MF 400: Art. No.: 60010

Accessory:

Wristband out of cotton, 75 cm long and 25 cm wide, Art.No.: 62001

The magnetic field therapy systems Magneto Fit MF 300 and MF 400 are proofed medical products in accordance to DIN EN 60601 – Made in Germany. That guarantees a high quality and the fulfillment of all the relevant safety standards. Manufacturer: Hane-Produktmanagement GmbH, D-86156 Augsburg
On general the magnetic field therapy is approved and used from the scientific medicine. We are obliged by law to point out, that there are existing alternative opinions regarding the effect, therapy success and therapy improvement in the scientific medicine.

Subject to technical changes. Rev. No. 11010911

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