# RhinoFit

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## **User manual**

## **Mode d'emploi**

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Deutsch.....	2
English.....	6
Français.....	10
Nederlands.....	14

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## IR-Light Anti allergy device

### RhinoFit®

successfully used by hay fever and allergic reactions to house dust and animal hair

# Tabel of Contents

Summary .....	7
Before operation .....	7
Safety instructions.....	7
Precautions / Contraindications.....	7
Applications .....	7
Frequency of application.....	8
Effectiveness .....	8
Operation of infrared light-therapy unit Rhino-Fit .....	8
Battery Replacement .....	8
Cleaning, maintenance and disposal .....	8
Guarantee.....	9
Technical specifications .....	9

## Before operation

Dear customer,

We enjoy that you bought a product from DAVITA® and congratulate you to your choice. In this user manual particularly the functions and possibilities of application of the RhinoFit are described in detail.

## Safety instructions

Before using the instrument the first time the safety instructions has to be read. Before switching on the instrument or operation proceed for a correct use as follows:

- Check visually if there is any sign of damage.
- Follow carefully all instructions given on the instrument.
- Remove any vessels containing liquids from the instrument, the instrument is not protected against penetration of liquids.

## Precautions / Contraindications

This natural treatment method is nearly without side effects. It is possible that the symptoms increase temporarily at the beginning of the application.

## Applications

Rhino Light therapy is one of the world, new and innovative form of light therapy. The term Rhino comes from the Greek meaning "nose". The DAVITA® infrared light therapy device RhinoFit works with 2 LED nose probes. Each probe is equipped with LEDs that deliver simultaneously a red light with a wavelength of 652 nm and an invisible infrared light with a wavelength of 940 nm. This process is protected by a patent. The device can be used successfully against:

### 1) The allergic rhinitis (hay fever)

Many inflammatory diseases of the nose and paranasal sinuses occur because of allergic reactions. The symptoms of "hay fever" in particular, runny nose, nasal itching, palatine itching, sneezing and nasal congestion.

### 2) The sensitivity to house dust and animal hair

The house dust mite allergy is caused by mites which are living in the dust of each house. The actual allergen is from the faeces of the mites.

An animal allergy is not triggered by the hair itself, but from the sebum, saliva or urine of each animal. These cling to the animal hair and are spread by them and the dust in the air.

When these allergens perch on the mucous membranes of the eyes, nose or bronchial tubes, they can trigger an allergic reaction. Thus, it can cause allergic symptoms such as watery eyes or itching -, cough, runny nose, sneezing, skin reactions and can lead to serious cases of respiratory distress and allergic asthma.

## Frequency of application

After a few treatments with the RhinoFit a significant improvement of symptoms may occur. The device can be used several times per day for every 3 minutes for about 2 weeks if necessary. The effect can be described as fast and long lasting. By repeated use the sensitivity to allergens will decrease, therefore it is possible that a definitive cure appears. If it is known when the exposure is expected with certain allergens then the RhinoFit can be used approximately 1 week before the start of the exposure.

## Effectiveness

An allergy means an excessive reaction of the immune system to certain foreign substances which are normally without any danger for the health. In the course of the allergic reaction the production of histamine is increased. This is a messenger of the allergic reactions of the body, such as itching, swelling of mucous membranes etc. Through the light-emitting diodes with a wavelength of 652 nm and a wavelength of 940 nm, the allergic symptoms and allergic nasal mucous membranes can be effectively reduced or even eliminated. The simultaneous use of red and infrared wavelength range can lead to particularly good results. During the application the temperature in the nose increases and the blood flow to the nasal mucosa is significantly increased. RhinoFit is an alternative or addition to drug therapy.

## Operation of infrared light-therapy unit RhinoFit

The operation of the RhinoFit is very simple.

- Below the unit is the battery case. It can be opened easily by simultaneously pressing and pulling a small lever on the bottom left of the battery cover. In the battery case, a 9 volt battery has to be inserted.
- The battery case can be closed easily again by inserting the cover and pressing the lever, which then snaps.
- On the top of the device a connector cable can be connected with the LEDs with the help of a plug. The LEDs are integrated in a mounting bracket for the nose.
- The LEDs in the form of a nose applicator have to be inserted into the nose. The probes should sit comfortable and tight so that they can not fall out of the nose.
- The blue button on the device must be pressed and the device is switched on. Thereby the unit beeps twice and the green LED on the right side of the blue button begins to shine. At the ends of the two nose probes a red light is visible with a specific wavelength of 652 nm. Also active is in each case a further LED which emits an infrared light with a wave length of 940 nm. This light is not visible for the human eye.
- After approx. 3 minutes the device beeps 8 times and then turns off. Now the application is completed.
- If the device has to be switched off during the 3 minutes, the blue button has to be pressed again.

## Battery Replacement

If the battery is low, the LED light starts to blink. In this case the battery has to be exchanged so that the light therapy can be performed with a good effect.

- Therefore the battery cover below the unit has to be opened, in which a small lever is simultaneously pressed and pulled on the left side of the cover.
- In the battery case the old battery has to be taken out and has to be replaced by a new one. It should be noted that the 9 volt block battery with regard to the + and – pole is installed correctly. The polarity is indicated on the inside of the case base.
- The battery case can be closed easily by inserting the cover and pressing the lever which then snaps.

## Cleaning, maintenance and disposal

A special maintenance for this device is not necessary. For cleaning the unit (surface of the housing) a clean damp cloth can be used. It should not come water into the housing and into the nose applicators. The nose applicators can be cleaned with a 70% alcohol-soaked cotton pad after the application.

For cleaning it should be dispensed of 100% alcohol solutions and organic solvents.

To increase battery life, it is advisable to remove the battery from the device after the application. Therewith a possible battery leakage is prevented.

Packaging: DAVITA® joint to the system „Grüner Punkt“. All parts of the packaging are recyclable materials and environmentally acceptable to dispose. Units which are signed with the crossed wheelie bin symbol are not allowed to throw into the domestic waste. There is the commitment to dispose such electro- and Electronic “Altgeräte” separately. The communal public authority which is responsible for the disposal gives information regarding the possibilities of a regulated disposal.

## Guarantee

DAVITA® guarantees for this instrument for a period of 24 months after date of sale. The guarantee is valid for material defects and manufacturing faults and includes the provision of spare parts as well as labour. Guarantee expires if the instrument has been damaged, or used improperly or unauthorized intervention was carried out. Keep the purchase receipt as proof and submit it together with your claim for guarantee. DAVITA® will only fulfil the guarantee upon receipt of your purchase proof. Wear parts, such as batteries and nose adapter are excluded from the guarantee.

## Technical specifications and further features

### Technical specifications and further features:

- For the application in the nose
- 2 rod-shaped nose applicators with holder and LEDs, which emit simultaneously a red light (652+/- 10 nm) and an infrared light (940 nm).
- Application duration: Approx. 3 minutes
- Operating temperature: 15°C – 40°C, humidity: 30% – 85%
- Storage temperature: -20°C – 60°C, humidity: 30% - 85%
- Dimensions of the unit: H x W x D: 78 x 60 x 31 mm
- Weight: 50 grams, without battery
- Operation with 9 volt block battery (not included in the scope of delivery)
- Scope of delivery:
  - Device including connection cable with nose applicators and holder
  - User manual
- Article number for the infrared light therapy unit RhinoFit: 30044, medical device with CE 0434
- Accessories for the RhinoFit: Nose adapter, Art.No.: 36001  
Should the device be used by several users, it is advisable for hygienic reasons to buy further nose adapters. So each user has a separate adapter.

Producer: Mesure Technology Co. LTD -TW Taipei  
DAVITA® Medizinische Produkte GmbH & Co. KG •P.O. Box 2004 •D-47518 Kleve  
Telephone: +49 (0) 1805 – 911 270 • Fax: +49 (0) 1805 – 911 275  
Email: [mail@davita.de](mailto:mail@davita.de) • [www.davita.de](http://www.davita.de)

Technical changes can be applied at any time without prior notice.. Rev. Nr.: 11051111, Art. No. : 005

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Fabrikant: Measure Technology Co. LTD -TW Taipei
DAVITA® Medizinische Produkte GmbH & Co. KG • Postfach 2004 • D-47518 Kleve